

Revisions			
Rev.	Description	Date	ECN. #
A	initial label release	07-06-2015	ECN5043
B	add UDI barcode and update address	09-08-2016	ECN5433



# I.V. ADMINISTRATION SET



**83" (211 cm) 10 drop Gravity Set  
with Roller Clamp,  
one MicroClave® Luer Locking injection site,  
one Pre-Pierced injection site and Spin Lock connector  
Priming Volume: 16 mL  
Qty: 1**

**REF 10-83011**

## INSTRUCTION FOR USE:

1. Remove spike protector, close roller clamp and insert into container.
2. Hang container, squeeze drip chamber till half full of fluid.
3. Remove male luer protector.
4. Open roller clamp, prime set to purge air from tubing, close roller clamp.
5. Attached male luer to venipuncture device.
6. Adjust fluid flow with roller clamp.

## CAUTION:

Single use disposable do not reuse.  
Replace per CDC guidelines and/or clinical policy.  
Dispose of as contaminated medical waste after use.  
Aseptically prepare injection site prior to use.



Do not use if package is damaged

**STERILE EO**

Store between -10°C and +40°C

**Rx  
ONLY**

**NEEDLE  
FREE**

**DO NOT  
RE-USE**

**DO NOT  
RE-STERILIZE**

**Not made with DEHP**

**Not made with natural rubber latex**

Distributed by Sarnova, HC, LLC's family of companies:  
Bound Tree Medical, LLC, DXE Medical, Inc.  
Emergency Medical Products, Inc. &  
Tri-anim Health Services, Inc.  
5000 Tuttle Crossing Blvd.  
Dublin, OH 43016  
855.287.2759  
www.curaplex.com

Made In China

MicroClave® is a registered trademark of ICU Medical, Inc.

**L6735 Rev. B**



GTIN(01): 00815277023016

Exp(17): YYYY-MM-DD

**LOT** LOT(10): MMMXXXPP

(01) 0081527702301 6 (17)YYMMDD (10)MMMXXXPP

**AMSINO**

Amsino International, Inc.  
708 Corporate Center Drive  
Pomona, CA 91768, USA  
<http://www.amsino.com>

	NAME	DATE	TITLE	
DRAWN			10-83011 each	
CHKD.				
APPR.			LABEL#: <b>L6735</b>	REV. <b>B</b>
1:1 ELECTRONIC FILE SCALE			SIZE: A4	PAGE 1 OF 1