



Distributed by Sarnova, HC, LLC's family of companies:
 Bound Tree Medical, LLC, DXE Medical, Inc.
 Emergency Medical Products, Inc. &
 Tri-anim Health Services, Inc.
 5000 Tuttle Crossing Blvd.
 Dublin, OH 43016
 855.287.2759
 www.curaplex.com

REF # **16380**



(01)00815277023887
 Radiotranslucent (Shadow)

LOT# Y090119-00



(10)Y090119-00
 Use By: 2019-09-01



(17)190901

Please visit www.boundtree.com for:
 • Current defibrillator compatibility
 • Pacing guidelines

Manufactured in the U.S.A.

REF: PEDIATRIC-ZOLL


Not compatible with all models. Check compatibility before use

Ten Pair - Pediatric (Infant) ≤ 10kg (22 lbs)

Monophasic or Biphasic Compatible
 Defibrillation ♥ Cardioversion ♥ Pacing ♥ ECG Monitoring
 Electrodes




Anterior-Anterior Placement

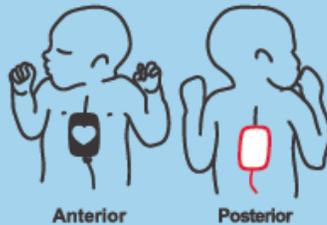

Anterior

Posterior

For Pediatric Use:
Manual Defibrillator:
 ≤ 10 kg (22 lbs)

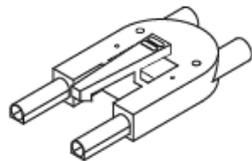Comparable to Zoll Multifunction Electrodes



Distributed by Sarnova, HC, LLC's family of companies:
 Bound Tree Medical, LLC, DXE Medical, Inc.
 Emergency Medical Products, Inc. &
 Tri-anim Health Services, Inc.
 5000 Tuttle Crossing Blvd.
 Dublin, OH 43016
 855.287.2759
 www.curaplex.com

REF # **16380**



(01)00815277023870
 Radiotranslucent (Shadow)

LOT# Y090119-00



(10)Y090119-00
 Use By: 2019-09-01



(17)190901

Please visit www.boundtree.com for:
 • Current defibrillator compatibility
 • Pacing guidelines


Manufactured in the U.S.A.

REF: PEDIATRIC-ZOLL

Not compatible with all models. Check compatibility before use

One Pair - Pediatric (Infant) ≤ 10kg (22 lbs)

Monophasic or Biphasic Compatible
 Defibrillation ♥ Cardioversion ♥ Pacing ♥ ECG Monitoring
 Electrodes

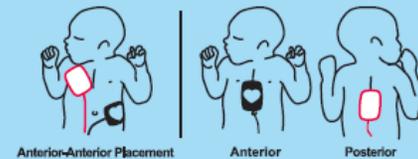

Anterior-Anterior Placement

Anterior

Posterior


For Pediatric Use:
Manual Defibrillator:
 ≤ 10 kg (22 lbs)



Instructions For Use:

1. Remove electrodes from pouch.
2. Ensure skin site is clean and dry.
3. Remove electrodes from release liner.
4. Apply electrodes firmly to patient's skin.
5. Connect electrodes to defibrillator.
6. Connect ECG leads for demand pacing.
7. **Caution - Do not exceed a setting of 100 joules while defibrillating.**
8. Apply a new set of electrodes after:
 ♥ 24 hours on skin ♥ 50 defibrillation shocks

Comparable to Zoll Multifunction Electrodes

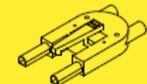

TEAR HERE

PEEL HERE

Package Artwork– Back

PEEL HERE

Warnings:

- Do not use if conductive polymer gel is dried or damaged.
- Misuse of electrodes may cause patient burns.
- Do not use if electrodes are damaged.

Cautions:

- Do not open package until immediately prior to using electrodes.
- Electrodes may dry out when removed from packaging and exposed to air.
- If electrodes do not adhere properly to patient, apply a new pair.
- Do not apply any substance to the skin surface that will leave residue.
- Do not apply electrodes over broken or damaged skin.
- Do not crush, fold or store package under heavy objects.
- Pacing requires separate leads and electrodes for monitoring.
- The adhesive foam may be a mild irritant to the skin.
- Electrode placement diagrams are suggestion only. Follow hospital protocol.
- Pacing, defibrillation or cardioversion may cause reddening of the skin.
- Do not exceed a setting of 100J for Pediatric (infant) electrodes

Storage Requirements:

To ensure maximum product performance, short term storage should not exceed 30 days at a maximum of +40° C (104° F), and a minimum of -20° C (-4° F).



RX ONLY
PRESCRIPTION ONLY



NO LATEX WAS USED IN THE MANUFACTURING OF THIS PRODUCT



WARNING/CAUTION



SINGLE USE ONLY

Comparable to Zoll Multifunction Electrodes



TEAR HERE