Nursing Anne

ENGLISH
ESPAÑOL
ITALIANO
FRANÇAIS
DEUTSCH
NEDERLANDS
NORSK
SVENSKA
SUOMI
CHINESE
JAPANESE
KOREAN

Directions for Use
Instrucciones de utilización
Istruzioni per l'Uso
Mode d'emploi
Gebrauchsanweisung
Handleiding
Bruksanvisning
Bruksanvisning
Käyttöohje
使用说明
取扱説明書
사용 방법

Table of Contents

Items Included	1
Skills Taught	1
Teeth	2
Carotid Pulse	2
Tracheostomy Plug & Care	2
NG Tube Placement	2
Intubation	2
Lungs	2
Stomach Reservoir	2
Belly Plates	2
Injection Pads	3
Waist, Knee and Ankle Joints	3
Arms	3
Legs	3
IV Arm	3
Replacing Skin & Vein System	3
SimPad Connection	4
Blood Pressure Arm	4
Auscultation of Heart, Breath, Bowel Sounds and Fetal Tones	4
Genitalia	4
Colon Reservoir	5
Urinary Catheterization and Enema Simulation	5
Troubleshooting	5
Attaching Nursing Anne Modules to Manikin	5
Care and Maintenance	6
Replacement Parts	6

Caution: Latex

This product contains **Natural Rubber** latex which may cause allergic reactions when in contact with humans.

Please contact Laerdal Customer Service

for more information on Replacement Parts in other skin tones.

The product, when carrying the CE-mark, is in compliance with essential CE requirements and other relevant provisions of council directive 1999/5/EC.

Items Included:

- (1) Full-Body Female Manikin
- (1) Articulating Blood Pressure Training Arm (Nursing Anne SimPad)
- (1) Articulating Female Multi-Venous IV Training Arm
- (1) Hospital Gown
- (1) Male Genitalia
- (1) Female Genitalia
- (3) Urinary Valves
- (3) Anal Valves
- (1) Simulated Blood Concentrate
- (1) 100 cc Syringe
- (1) Manikin Lubricant
- (1) Assembly Tool Kit

Wig pictured on front cover, not included

Optional Accessories

Nursing Anne Modules:

- (1) Breast Exam Module, with interchangeable abnormalities
- (1) Mastectomy Module with staples and drain
- (1) Fundus Module with interchangeable uteri

Wound Care & Assessment Set:

- (1) Abdominal Incision Module with Painted Sutures
- (1) Abdominal Incision Module with Staples and Penrose Drain
- (1) Abdominal Incision Module with Nylon Sutures and Penrose Drain
- (1) Abdominal Subcutaneous Heparin and Insulin Injection Module
- (1) Abdominal Packing Module
- (1) Infected Colostomy Stoma
- (1) Vento-Gluteal and Gluteal Decubitus Ulcer Modules
- (1) Below Knee Amputation Stump
- (1) Thigh Packing and Irrigation Module
- (1) Thigh Suture Module
- (1) Thigh Debridement Module
- (1) Varicose Vein Leg with Stasis Ulcer
- (1) Diabetic Foot Module

Skills Taught:

- Basic patient handling
- Denture care
- Oral hygiene
- Oral and nasal Intubation
- Eye and ear irrigation (simulated)
- NG Tube insertion, care medication administration and removal
- Lavage/Gavage
- Tracheostomy care and suctioning
- Blood pressure skills (When used with SimPad)
- IV care and management
- Subcutaneous and intramuscular injection
- Oxygen delivery procedures
- Ostomy irrigation and care
- Catheterization skills
- Enema simulation
- Colonic irrigation
- Wound assessment and care
- Bandaging and dressing
- Post-surgical mastectomy-care procedures
- Recognition of breast disorders, their sizes and relative locations
- Practice of fundus massage skills
- Auscultation and recognition of normal and abnormal heart, breath and bowel sounds and fetal tones (When used with SimPad)

Nursing Anne Advanced is a full-body, lifelike vinyl manikin designed to teach all skills from basic patient handling to advanced nursing, including the measurement of noninvasive blood pressure and the auscultation and recognition of normal and abnormal heart, lung and bowel sounds and fetal tones.

The manufacturing quality of this simulator should provide many sessions of training when reasonable care and maintenance are practiced.

Laerdal Recommends:

IV Injections – 22 gauge needle, or smaller

Tracheostomy Tube – Size 6

NG Tube – Size 16 French

Endotracheal Tube – 7.5 or smaller

Laryngoscope Blade - Size #2 or #3, straight or curved blade

Urethral Catheter – 16 French

Enema Simulation – 7 mm

Teeth –

To remove: Grasp firmly and lift.

Insert and remove bottom teeth first when performing denture care.

To reinsert both sets:

- Begin on one side and work around, aligning ridges with corresponding grooves in jaws.
(Dusting with baby powder is helpful.)
- Press firmly into place.

Carotid Pulse –

To generate a carotid pulse, use the hand held red bulb located on the right axillary side of manikin. Firmly squeeze bulb while palpating for pulse at carotid site.

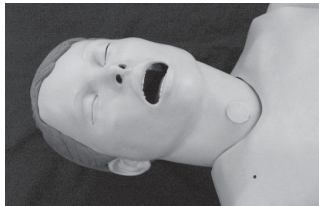
Photo 1

Tracheostomy Plug –

The tracheostomy plug may be removed by grasping firmly, then lifting up and out. To replace, press into hole. (Photo 1)

Tracheostomy Care –

- Mix solution of 1/2 cup mild liquid detergent and 1/2 cup water.
- Remove lungs.
- Pour mixture into simulated lungs.
- Fill lungs to a level where suction catheter will pick up mixture.
- Reattach lungs at connector.

The consistency of the solution approximates the mucus normally suctioned in a tracheostomy patient.

We recommend a size 6 tracheostomy tube. Opening will accommodate other sizes, but a size 6 is usually most suitable.

Drain and air-dry lungs immediately after each use.

NG Tube Placement –

This manikin features an esophagus and stomach reservoir for practice of NG tube insertion and skills such as lavage and gavage.

For best results, lubricate the tube with either silicone spray or liquid detergent prior to NG or OG tube insertion.

Intubation –

The following equipment is recommended:

- Endotracheal tube, size 7.5 ID or smaller
- Manikin Lubricant or liquid soap
- 10 ml syringe
- Laryngoscope blade, size #2 or #3 straight or curved.
- Laryngoscope handle
- NG Tube, size 16 French

Remove trach plug prior to insertion of intubation tubes.

Intubation tubes and airway passages should be sprayed with manikin lubricant or lubricated with liquid soap prior to intubation.

Lungs –

Lungs may be removed from bronchial tubes by unscrewing them at the connector site. To replace, reverse procedure.

Stomach Reservoir –

Stomach reservoir attaches to esophagus with a connector. To fill reservoir with water, use 100 cc syringe.

Dusting with talcum powder is helpful in replacing belly plates.

Belly Plate with Multi-Sounds –

To remove:

- Apply pressure to center of plate.
- Work your fingers under edge of plate while lifting.
- Disconnect bowel sound speaker connector to release plate.

To insert:

- Position plate over cavity.
- Reconnect bowel sound speaker.
- Work plate edges into groove on pelvis cavity.

Belly Plate with Colostomy Stoma –

To remove:

- Apply pressure to center of plate.
- Work fingers under edge while lifting.
(Photo 2)

Photo 2

To insert:

- Attach valve and genitalia.
- Work plate edges into pelvis cavity.

The belly plate with colostomy also contains the urinary reservoir and colostomy stoma.

To fill reservoir:

- Remove colostomy stoma.
- Fill with fluid.
- Gently reinsert stoma.

Make sure genitalia and urinary connectors are in place prior to filling.

Injection Pads –

- To remove, squeeze pad with fingers and pull.
- To replace, squeeze pad and insert in opening.

Dusting with talcum powder is helpful in replacing belly plates.

Injection pads may be injected with water. Foam should be removed from pads immediately following training. Squeeze out fluid and air dry.

Waist, Knee and Ankle Joints –

These joints are connected with a bolt and lock nut. To remove, unscrew lock nut and pull bolt from joint. Segments will now easily disconnect.

Attachment hardware is located in envelope inside carton.

Arms –

Arms are attached with a 4 1/2" long bolt, three washers, a spring and a wing nut.

To attach:

- Unfasten chest skin from shoulders.
- Pull skin back to expose chest cavity.
- Locate hole in shoulder.
- Remove deltoid injection pad from arm by squeezing skin in center of pad while pulling.
- Slide small washer over bolt.
- Insert bolt through hole in arm and hole in manikin's shoulder.
- Add large washer; spring, another large washer and wing nut to bolt inside manikin's chest. (Figure 1)
- Use a screwdriver to hold bolt steady while tightening wing nut.

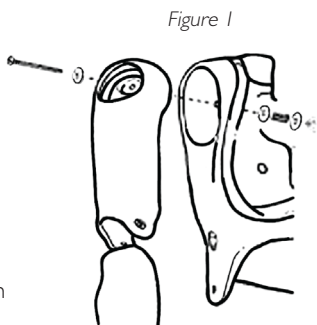
Figure 1

To remove arms, reverse procedure.

Legs –

Legs are attached with a 6" long bolt, a spring, a large washer, a small washer and a wing nut.

Attachment hardware is located in envelope inside carton.

To attach:

- Remove belly plate and colon reservoir from manikin to locate hole in hip.
- Remove thigh injection pad from leg by squeezing skin in center of pad while pulling.
- Slide spring and small washer over bolt.
- Insert bolt through hole in thigh.
- Insert bolt through hole in pelvis.

- Slide large washer and wing nut over bolt.
- Use screwdriver to hold bolt while tightening wing nut inside pelvis. (Figure 2)

To remove legs, reverse procedure.

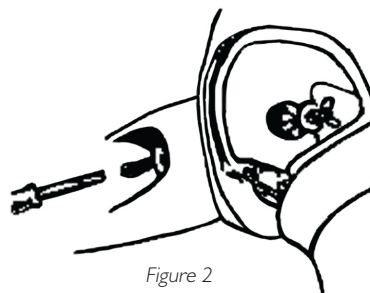
Figure 2

IV Arm – Multiple Venipuncture sites:

- Dorsal Veins of Hand (3)
- Antecubital
 - Cephalic Vein
 - Median Vein
 - Basilic Vein

A 22 gauge needle or smaller is recommended to extend the life of the IV Arm.

When using an IV catheter, lubricate with manikin lubricant for easier insertion.

Directions for Use:

- Attach IV bag to IV tubing.
- Attach IV tubing to either vein.
- Allow fluid to flow through arm and out other vein.
- Clamp off flow of water from open vein.

The arm is now ready to practice venipuncture.

When excessive leaking occurs at puncture sites, either a new vein system or skin should be installed to reduce loss of fluid.

We recommend working at a sink when replacing the skin and vein system.

Replacing Skin and Veins:**1. Remove skin:**

If replacing skin and veins, cut off skin. This can be done with a sharp knife, scalpel or scissors. (Figure 3) When finished, discard skin. (Figure 4)

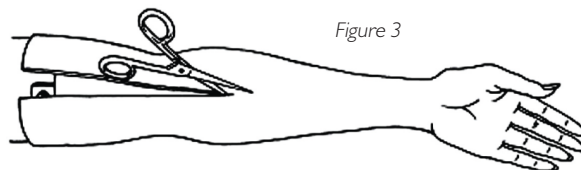
Figure 3

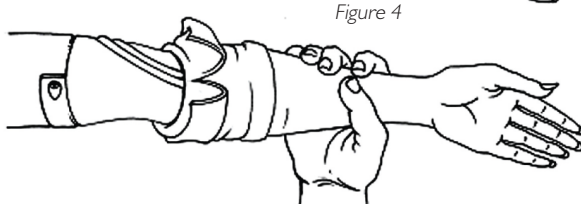
Figure 4

Replacing veins: (Keeping skin)

1. Lubricate inside of skin with liquid detergent; let it flow down into the finger area.
2. Begin at top of arm and slowly pull skin down and off of arm. Do not roll, as that will cause skin to bind.
3. Remove tubing from track in mandrel. Glue may need to be scraped away to allow removal.
4. Rinse and dry vein grooves well and swab with alcohol. Be sure to remove any excess glue.
5. Place new veins along grooves, (Figure 5) spot gluing as needed. (We recommend a fast-drying glue.)

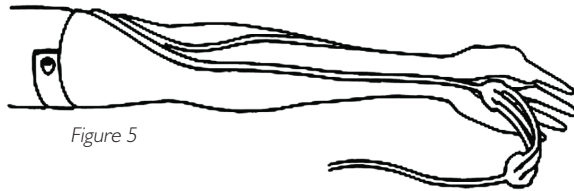
Figure 5

6. Generously lather arm mandrel with liquid soap.

*Heat arms skins before replacing on mandrel.
This can be done with a blow-dryer.*

7. Slide hand into skin.
8. Work skin over fingers, as with a glove.
9. Work arm skin up, over mandrel.

SimPad Connection

1. Connect Nursing Anne to Link Box, via cable located on back of manikin. (Photo 3). For some manikins the adapter cable supplied with SimPad may have to be used.
2. Connect clear tubing exiting manikin together with the cable.

Photo 3

See SimPad DFU for complete Heart, Breath and Bowel Sound Auscultation.

Blood Pressure Arm –

1. Place Blood Pressure Cuff on arm.
2. Attach tubing on cuff to clear tube, located underneath Nursing Anne's arm. (Photo 4)

Photo 4

See SimPad DFU for complete Blood Pressure operating procedures.

Do not insert needles into blood pressure arm.

Auscultation of Heart, Breath and Bowel Sounds and Fetal Tones – See SimPad DFU for complete operating procedures.

Genitalia:

Both male and female genitalia have been provided for urinary catheterization and enema training procedures.

Tube-like valves, with a screw cap on one end, connect urethra and anus to corresponding reservoirs. This cap must be connected to genitalia openings. Soft vinyl end of connector is attached to urinary and anal reservoirs with a white clamp. (Long valve connects urinary reservoir; short valve connects anal reservoir.)

Pelvic support pin must be removed and replaced immediately when assembling and disassembling genitalia. (Figure 6)

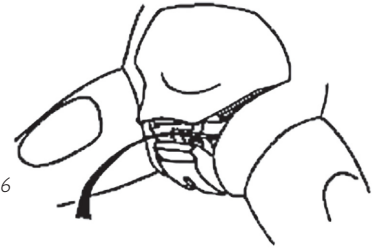
Figure 6

Pelvic support pin must be replaced.

Failure to replace pin will result in pelvic spread.

If this occurs, the genitalia module will no longer fit properly in the manikin.

To disassemble:

1. Lift up on belly plate.
2. Pull upper portion of genitalia back to remove pelvic support pin.
3. Belly plate, genitalia with connectors, and colon reservoir may now be removed simultaneously.

To assemble: (Photos 5, 6 & 7)

1. Screw both valves onto genitalia. When attaching valves, be sure alignment of genitalia, urinary reservoir (belly plate) and colon reservoir are as illustrated in Figure 7.
2. Lift up on edge of belly plate.
3. Pull down upper portion of genital.
4. Insert into pelvic cavity.
5. Replace the pelvic support pin.
6. Press belly plate into position.

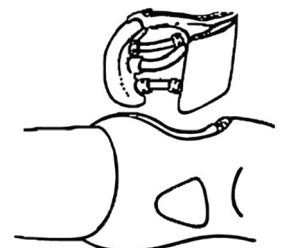
Figure 7

*Long valve connects urinary reservoir;
short valve connects anal reservoir;*

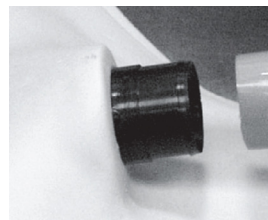
Photo 5
Slide tube to meet ridge on connector.

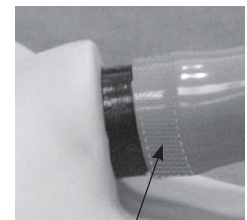
Photo 6
Area where clamp is placed.

Photo 7
Apply clamp and squeeze to tighten.

Colon Reservoir –

Colon reservoir inserts into pelvic cavity with narrow end downward and connector pointing outward. This connector attaches reservoir to anal valve on genitalia. (Photo 8)

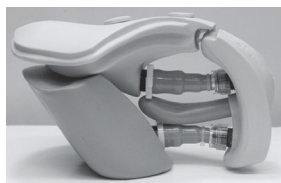
Photo 8

Reservoir should be removed from manikin, inverted, drained and completely air-dried before storing.

Urinary Catheterization and Enema Simulation –

Catheters should be well lubricated with manikin lubricant or full strength liquid soap prior to using.

After several uses, catheter may “hang” when water has washed away lubricant, causing friction and binding. If this occurs, generously lubricate a catheter and insert into urethra several times.

If catheter hangs during removal, work it with an “in and out” motion. Use warm water in reservoirs when possible to keep soap from clogging the catheters. Make sure reservoirs are well drained of all water before disconnecting valves.

Check List for Valves That May Leak:

- a. Do valve connections have an O-ring?
- b. Does the white clamp fit firmly against base of valve?
- c. Are valves attached correctly?
- d. If valves are attached correctly, are they properly connected to reservoirs and to genitalia?

A size 16 French catheter is suggested for urethral catheterization.

Size 7mm catheter is suggested for enema simulation.

Troubleshooting –**What can I do if my manikin develops a pelvic spread?**

Cinch pelvis with a belt and heat with a hair dryer to soften, cinching as you heat. Insert pin, leaving belt in place until cool.

What do I do if my connectors leak?

Make sure C-clamps are squeezed tightly into place and connectors are screwed on tightly.

What can I do if I can't remove my catheter?

Make sure the cuff is deflated.

Attaching Nursing Anne Modules to Manikin:

Fundus Skills Module - Legs, internal components and genitalia must be removed from pelvis.

Applying baby powder to skin and modules will assist in attachment and removal.

To remove legs:

1. Remove abdominal plate by pulling up on its edges.
2. Remove genitalia and all internal attachments, reservoirs and pelvic pin. (If using sounds abdominal plate, detach cable.)
3. Remove thigh injection pad by squeezing skin in center of pad while pulling.
4. Locate wing nut on inside of pelvis.
5. Using screwdriver to hold bolt in place, twist and remove wing nut.
6. Remove spring and small washer from bolt. (Figure 8)
7. Remove bolt from thigh and pelvis.

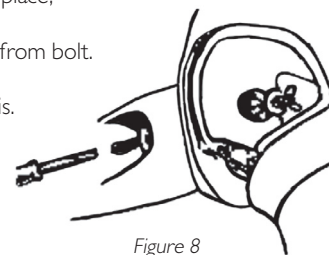
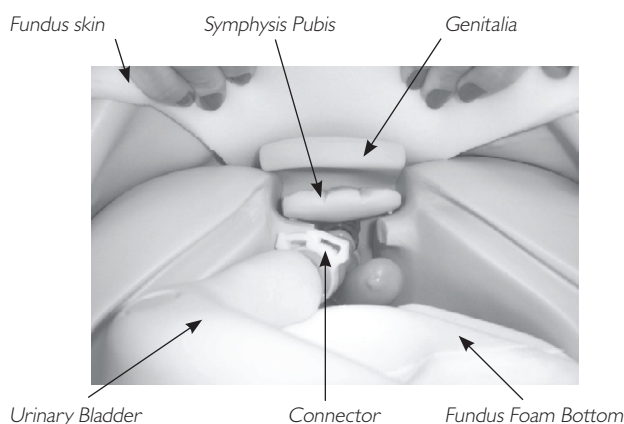
Figure 8

To attach Fundus Module:

1. Apply baby powder to pelvis and inner fundus skin.
2. Remove silicone straps from cut portion at back of fundus skin.
3. Slide skin up and over pelvis, with slit and straps positioned on posterior side of manikin.
4. Reattach legs. (Legs must be reattached at this point, without proceeding further with fundus upgrade assembly.) When reattaching legs, make sure to leave space in between thigh and skin to allow for leg articulation without rubbing against fundus skin.
5. Pull fundus skin partially down pelvis, exposing internal pelvic area.
6. Place fundus foam bottom inside pelvis.
7. Using connectors, connect genitalia to urinary reservoir.
8. Place genitalia/urinary reservoir inside pelvis. This must be done from the genitalia area. Go through the genitalia opening to place the reservoir and genitalia. (Photo 9)
9. Place symphysis pubis pelvic pin. This replaces the pelvic pin already located in the manikin. (Photo 9)
10. Pull fundus skin up and attach three (3) tabs at back.
11. Insert “firm” or “boggy” fundus ball.

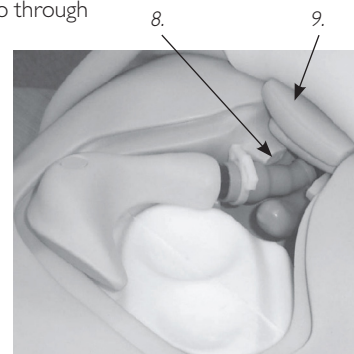
Photo 9

Mastectomy Module:

1. Remove chest skin from female manikin by detaching at shoulders and back of torso.
2. Place mastectomy module with single foam breast insert over torso.
3. Reattach at shoulders and back of torso.

Breast Exam Module:

Caution!
Misuse could result in damage to this module.

1. Remove chest skin from female manikin by detaching at shoulder and along back of torso.
2. Apply powder to back of nodules, back of breasts and inside tunnels. Nodules with pegs are to be fitted in the chest plate; nodules with out pegs may be inserted into tunnels on back of breasts. **Do not use sharp instruments to insert or remove nodules from breasts!**
3. Position breasts on plate. Nodules on back of breast will align with grooves in chest plate when positioned correctly.
4. Reattach chest skin.

Care and Maintenance –

1. Clean with mild soap and water; do not submerge manikin or parts in cleaning fluids or water. Apply a light coat of talcum powder to the face and chest skin to achieve skin-like feeling.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, each manikin should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed. After use of injection pads (use water only), accumulated water should be squeezed out. Do not store wet foam pads in the skin. To prevent mildew or mold, pads can be soaked in a mild solution of disinfectant and water or bleach and water. Squeeze excess solution from pads, allow them to dry, then store or reinsert in manikin.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.

Replacement Parts:

Product Number:	Description:
380471	Plug, Belly plate-Adult
380475	Pin, Pelvis
380600	Leg Assembly, Left Adult
380650	Leg Assembly, Right Adult
381402	Pad Assembly, Deltoid Injection Adult
381403	Stoma, Infected Colostomy
381404	Stump, Left Lower Leg
381410	Foot, Diabetic Right Adult
381411	Pad, Thigh Packing Adult
381412	Pad, Thigh Sutured Adult
381413	Leg, Varicose Lower Right Adult
381417	Pad, Thigh Debridement Adult
200-02050	Cuff Assembly, Blood Pressure
300-00150	Pad Assembly, Thigh Injection Adult
300-00450	Valve/Clamp Set, Anal Adult Hosp Urin/Anal CVK
300-00550	Valve/Clamp Set, Urinary Hosp Adult
300-00650	Lubricant, Manikin-4 oz Bottle W/Sprayer
300-00750	Blood, Simulated-4 oz Red
300-00850	Plug, Tracheostomy
300-00950	Denture Set, Upper/Lower Adult
300-01850	Teeth, Upper-Adult
300-01950	Teeth, Lower-Adult
300-02050	Belly plate Assembly, Hosp w/Speakers Adult
300-02150	Belly plate Assembly, Hosp 3 Hole Adult
300-02250	Reservoir Assembly, Colon Hosp Adult
300-02950	Thigh, Right Adult
300-03050	Leg, Lower Right Adult
300-03150	Foot, Right Adult
300-03250	Thigh, Left Adult
300-03350	Leg, Lower Left Adult
300-03450	Foot, Left Adult
300-03550	Belly plate, Packing Adult
300-03650	Belly plate Sutured Adult
300-03850	Genitalia Assembly, Adult Male
300-03950	Genitalia Assembly, Adult Female
300-04250	Hardware Set, Adult Manikin-Legs/Arms to Body
300-04350	Hardware Set, Adult Manikin-Pelvis/Torso
300-04550	Belly plate Assembly, Suture/Drain Hosp Adult
300-04850	Gown, Hospital-Back Tie
300-04950	Stoma, Soft Style w/Thru Hole
325-00150	Pad Assembly, Ventral Gluteal Injection Adult Female
325-00250	Pad Assembly, Gluteal Injection Adult Female
325-00350	Kit, Skin/Vein Adult Female Multi-Venous IV Arm
325-00450	Module, Fundus Skills and Assessment-Nursing Anne
325-00550	Wound Set, Care and Assessment Adult Female
325-00650	Module, Mastectomy-Nursing Anne
325-00750	Module, Breast Exam-Nursing Anne
325-00850	Head Assembly, Female-Advanced Intubation Head
325-00950	Lung Assembly, Small-Adult Intubation Head
325-01150	Genitalia, Blank-Adult Female
325-01250	Arm Assembly, Blood Pressure
325-01350	Arm Assembly, Right Plain Adult Female
325-01450	Arm Assembly, Left Plain Adult Female
325-01550	Arm, Upper Left Plain Female
325-01650	Forearm, Left Plain w/Hand Female
325-01750	Arm, Upper Right Plain Female
325-01850	Forearm, Right Plain w/Hand Female
325-01050	Chest plate Assembly, Hospital Adult Female
325-02050	Skin Assembly, Chest-Hospital Adult Female
325-02150	Insert, Breast Foam-Left
325-02250	Insert, Breast Foam-Right
325-02350	Pad, Dorsal Gluteal Ulcer Adult Female
325-02450	Pad, Ventral Gluteal Ulcer Adult Female
325-02550	Torso Assembly, Hospital Adult Female
325-02650	Pelvis, Hospital w/o Bridge Adult Female
325-02750	Reservoir Assembly, Stomach Hospital Adult Female
325-03650	Chest Skin (No ECG)
325-03450	Chest Skin (Spacer Style)
325-03550	Retrofit Kit (Chest Skin Spacer Style)
375-51001	Arm, Female Multi-Venous IV Training

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제품 번호:	설명:
380471	Plug, Belly plate-Adult
380475	Pin, Pelvis
380600	Leg Assembly, Left Adult
380650	Leg Assembly, Right Adult
381402	Pad Assembly, Deltoid Injection Adult
381403	Stoma, Infected Colostomy
381404	Stump, Left Lower Leg
381410	Foot, Diabetic Right Adult
381411	Pad, Thigh Packing Adult
381412	Pad, Thigh Sutured Adult
381413	Leg, Varicose Lower Right Adult
381417	Pad, Thigh Debridement Adult
200-02050	Cuff Assembly, Blood Pressure
300-00150	Pad Assembly, Thigh Injection Adult
300-00450	Valve/Clamp Set, Anal Adult Hosp Urin/Anal CVK
300-00550	Valve/Clamp Set, Urinary Hosp Adult
300-00650	Lubricant, Manikin-4 oz Bottle W/Sprayer
300-00750	Blood, Simulated-4 oz Red
300-00850	Plug, Tracheostomy
300-00950	Denture Set, Upper/Lower Adult
300-01850	Teeth, Upper-Adult
300-01950	Teeth, Lower-Adult
300-02050	Belly plate Assembly, Hosp w/Speakers Adult
300-02150	Belly plate Assembly, Hosp 3 Hole Adult
300-02250	Reservoir Assembly, Colon Hosp Adult
300-02950	Thigh, Right Adult
300-03050	Leg, Lower Right Adult
300-03150	Foot, Right Adult
300-03250	Thigh, Left Adult
300-03350	Leg, Lower Left Adult
300-03450	Foot, Left Adult
300-03550	Belly plate, Packing Adult
300-03650	Belly plate Sutured Adult
300-03850	Genitalia Assembly, Adult Male
300-03950	Genitalia Assembly, Adult Female
300-04250	Hardware Set, Adult Manikin-Legs/Arms to Body
300-04350	Hardware Set, Adult Manikin-Pelvis/Torso
300-04550	Belly plate Assembly, Suture/Drain Hosp Adult
300-04850	Gown, Hospital-Back Tie
300-04950	Stoma, Soft Style w/Thru Hole
325-00150	Pad Assembly, Ventral Gluteal Injection Adult Female
325-00250	Pad Assembly, Gluteal Injection Adult Female
325-00350	Kit, Skin/Vein Adult Female Multi-Venous IV Arm
325-00450	Module, Fundus Skills and Assessment-Nursing Anne
325-00550	Wound Set, Care and Assessment Adult Female
325-00650	Module, Mastectomy-Nursing Anne
325-00750	Module, Breast Exam-Nursing Anne
325-00850	Head Assembly, Female-Advanced Intubation Head
325-00950	Lung Assembly, Small-Adult Intubation Head
325-01050	Chest plate Assembly, Hospital Adult Female
325-01150	Genitalia, Blank-Adult Female
325-01250	Arm Assembly, Blood Pressure
325-01350	Arm Assembly, Right Plain Adult Female
325-01450	Arm Assembly, Left Plain Adult Female
325-01550	Arm, Upper Left Plain Female
325-01650	Forearm, Left Plain w/Hand Female
325-01750	Arm, Upper Right Plain Female
325-01850	Forearm, Right Plain w/Hand Female
325-02050	Skin Assembly, Chest-Hospital Adult Female
325-02150	Insert, Breast Foam-Left
325-02250	Insert, Breast Foam-Right
325-02350	Pad, Dorsal Gluteal Ulcer Adult Female
325-02450	Pad, Ventral Gluteal Ulcer Adult Female
325-02550	Torso Assembly, Hospital Adult Female
325-02650	Pelvis, Hospital w/o Bridge Adult Female
325-02750	Reservoir Assembly, Stomach Hospital Adult Female
325-03650	Chest Skin (No ECG)
325-03450	Chest Skin (Spacer Style)
325-03550	Retrofit Kit (Chest Skin Spacer Style)
375-51001	Arm, Female Multi-Venous IV Training

20-05824

© 2012 Laerdal Medical/All rights reserved
20-05824 Rev C Printed in US

Laerdal

helping save lives