


SimPad SkillReporter

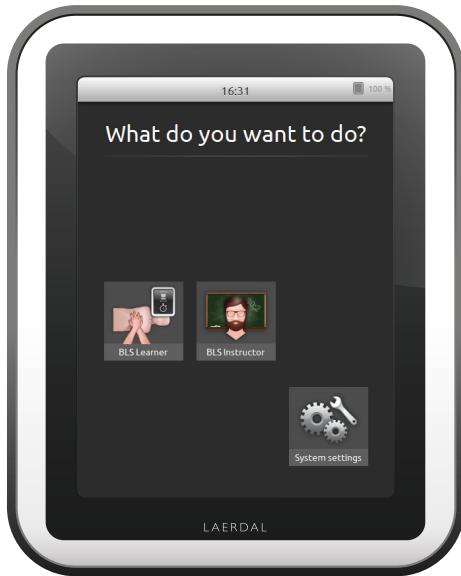

User Guide

INTRODUCTION

SimPad is a wireless touchscreen device for use with Laerdal manikins. This version is installed with SkillReporter software for Basic Life Support (BLS) training. To get feedback on the CPR performance, use QCPR enabled manikins, either Resusci Anne or Resusci Baby. SkillReporter has two modes, BLS Learner and BLS Instructor:

The screenshot shows the SkillReporter app interface. At the top, the status bar displays the time 16:31 and 100% battery. The main heading is "What do you want to do?". Below this, there are three main options:


- BLS Learner**: Represented by an icon of hands performing CPR. A callout box explains: "BLS Learner gives feedback on CPR done on a single manikin."
- BLS Instructor**: Represented by an icon of a man with a beard and glasses. A callout box explains: "BLS Instructor enables the instructor to provide AED training. Feedback can be displayed from up to 6 manikins."
- System settings**: Represented by an icon of gears and a wrench. A callout box explains: "Adjust brightness, clock, and more."

A bracket groups the BLS Learner and BLS Instructor options under the label "SkillReporter".

CONTENT

1	Introduction	2
	Content	3
	What's in the box	4
2	Get Started	5
	Charge the battery	5
	Startup	5
3	Connect	6
	Connectors	6
	Connecting manikins	6
	Connect to one manikin using USB cable	7
	Connect using WiFi	7
	Select manikins	8
	Set up and select network	9
	Laerdal Router	9
	Edit manikin name	10
	Can't find a manikin?	10
4	SimPad SkillReporter	11
	How to use SimPad SkillReporter	11
	BLS Learner	12
	BLS Instructor	13
	Choose algorithm	14
	Manage Learners	15
	Event registration	17
	Defibrillator features	18
	Live feedback	20
	Timeline	21
	Results	22
	BLS Settings	25
	Get manikin information	26
5	Manikin Error State	27
	Red light on Resusci Anne Manikin	27

WHAT'S IN THE BOX

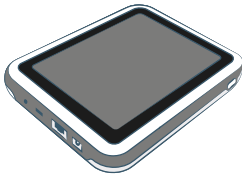


SimPad



SimPad Sleeve



SimPad to Manikin Cable

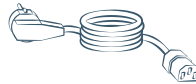

Power Cable



SimPad to PC Cable



Power Adaptor



User Guide for SkillReporter



Important Product Information


CHARGE THE BATTERY

Connect SimPad to the AC adapter and plug it into a wall outlet.

When the battery charges, the yellow LED flashes.

When the battery is fully charged, the yellow LED glows steadily.

If you start SimPad when the battery level is very low, the red LED flashes.

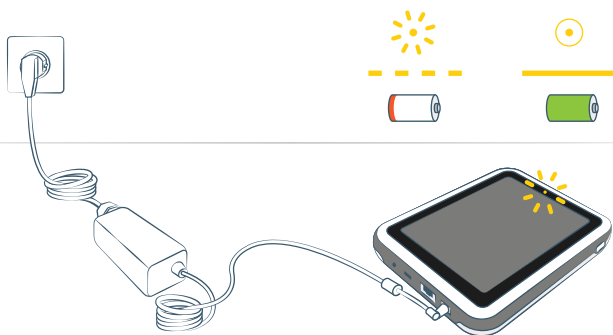


STARTUP

Press and hold down the Power button for half a second to switch on the SimPad. The first startup can take a few minutes. You are prompted to set the time, date, language, and device name.



CONNECTORS

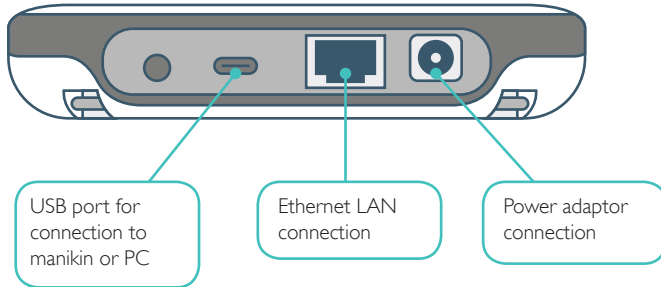


CONNECTING MANIKINS

There are several ways to connect SimPad to the manikins. When connecting SimPad to only one manikin, use either the USB cable or WiFi. No router is needed.

When connecting SimPad to more than one manikin (up to 6), use a WiFi connection with a router. We recommend using the Laerdal Router as it simplifies the setup and improves stability.


CONNECT TO ONE MANIKIN USING USB CABLE

The USB cable provides the most reliable connection when connecting SimPad to one manikin. When using the USB cable, the manikin is powered by SimPad. To avoid a quick battery discharge when the manikin is powered by SimPad, we recommend you plug in the power adaptor:

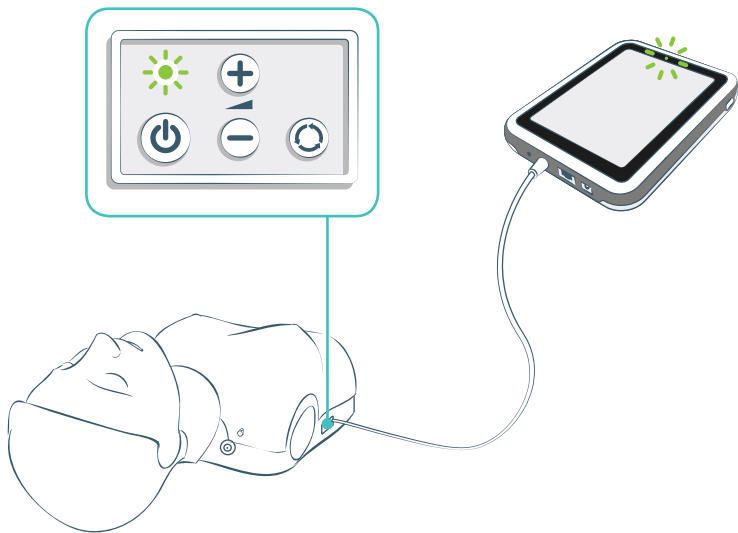


CONNECT USING WIFI

For wireless use, make sure the manikin is in WiFi mode, indicated by the green WiFi symbol on the manikin's control panel. The toggle button toggles between cable, WiFi, and Bluetooth mode.

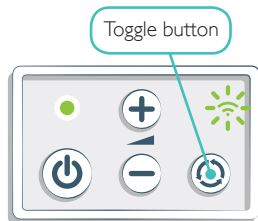


SELECT MANIKINS

If you start a session without having manikins connected, you are prompted to select manikin(s). In **BLS Learner** you can only select one manikin. This happens automatically if the manikin is connected through the USB cable, or if only one manikin is available. In **BLS Instructor** you can choose up to 6 manikins.

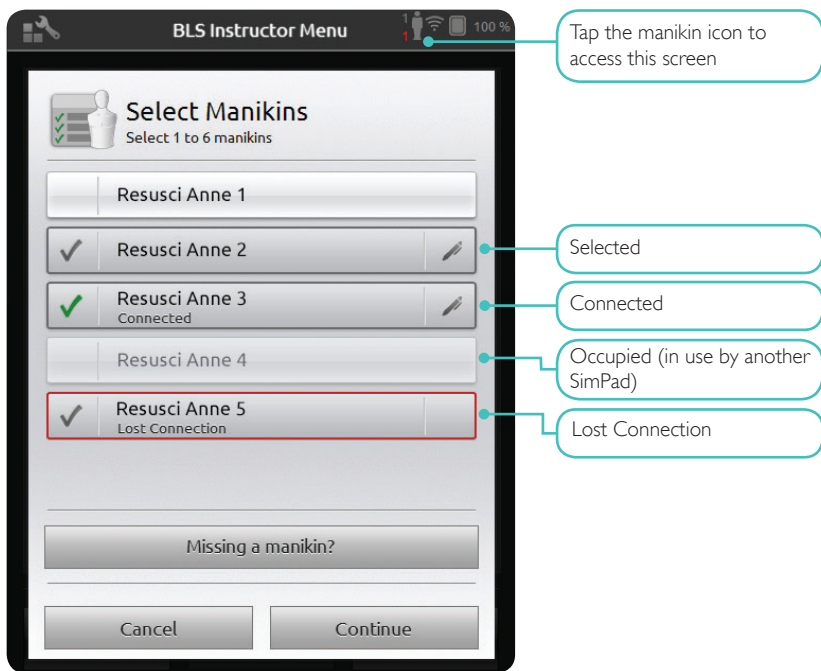


SET UP AND SELECT NETWORK


If you have selected more than one manikin (**BLS Instructor**), you are prompted to choose a network. The network enables communication between several manikins and SimPad. We recommend that you use the Laerdal Router (purchased separately) to establish the network.

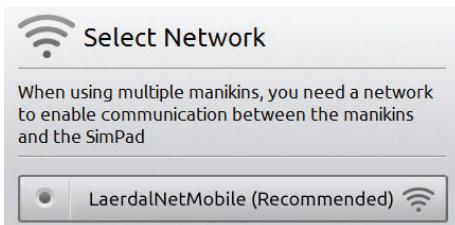

LAERDAL ROUTER

Let the router start up completely (approx. 2 minutes) before starting up SimPad and the manikins. Also make sure the router switch is set to Router or AP, not Client. For optimal performance, place the router above floor level. Do not reset or otherwise change the router (i.e. password, configuration etc). To reboot, simply disconnect and reconnect the power. To power the router; use only the included USB power supply (5V). Using the SimPad power supply (12V) may cause permanent damage to the router. A network from a Laerdal Router is listed as LaerdalNetMobile.

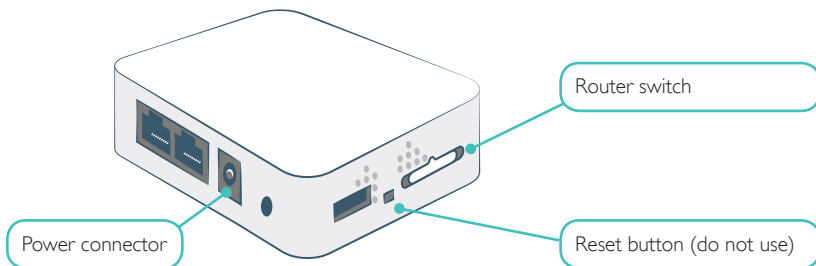


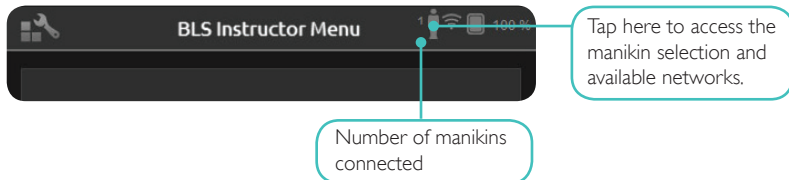

EDIT MANIKIN NAME

The default manikin name is **Resusci Anne** or **Resusci Baby**. To keep track of multiple manikins, give each one a unique name.

TIP: Switch on one manikin at the time and tap Edit name when the manikin appears on the screen. Type a new name and put a corresponding name tag on the manikin. Repeat this process for every new manikin.

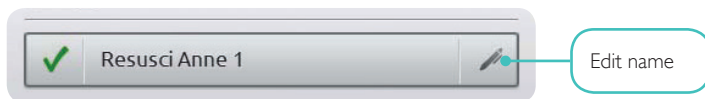

CAN'T FIND A MANIKIN?

If the manikin you want to use doesn't appear on the list, make sure the manikin is switched on and in WiFi mode, indicated by the green WiFi symbol on the control panel on the manikin. Be patient - it might take up to 15 seconds to identify the manikin. If the manikin still doesn't appear, it may be connected to a different network than SimPad. To make it available for selection, press and hold the manikin's toggle button. After two seconds a beep confirms that the manikin is reset. About 15 seconds later the manikin appears on the list.

Connection problems may occur in areas with e.g. high WiFi traffic.

The most reliable connection for single manikin use is the USB cable. A cable connected manikin appears in the manikin list as preselected and connected.

HOW TO USE SIMPAD SKILLREPORTER


SkillReporter has two modes optimized for learner and instructor use:



BLS Learner



BLS Instructor

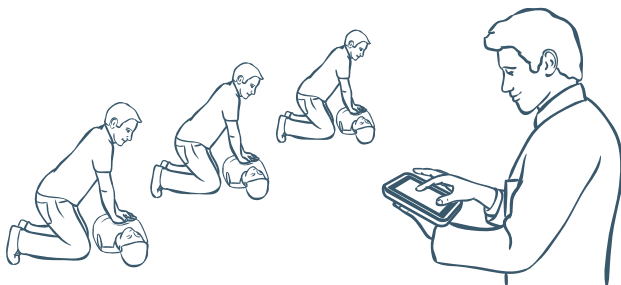

BLS LEARNER

BLS Learner gives feedback on CPR performance for learners using a single manikin. You can choose between **Practice** and **Assessment**, and between **CPR (single rescuer or two-rescuer)**, **Compressions Only**, and **Ventilations Only**.

The screenshot shows the BLS Learner Menu interface. At the top, it says "BLS Learner Menu" and shows a battery level of 100%. The menu is divided into two main sections: "Practice" and "Assessment".


- Practice:** This section features an illustration of hands performing CPR on a manikin. A callout box points to this section with the text: "Practice CPR with corrective feedback."
- Assessment:** This section also features an illustration of hands performing CPR on a manikin. A callout box points to this section with the text: "Test your performance (with or without feedback) and save your results."

At the bottom of the menu, there are three buttons: "Quit", "Last Session", and "Results".

- A callout box points to the "Last Session" button with the text: "View results from the previous session."
- A callout box points to the "Results" button with the text: "View results from the saved sessions."

BLS INSTRUCTOR

In **BLS Instructor** the interface is designed to help the instructor review learners' performance on one or several manikins, register events, and control defibrillation features*. Sessions can be saved and features for managing learners are available.

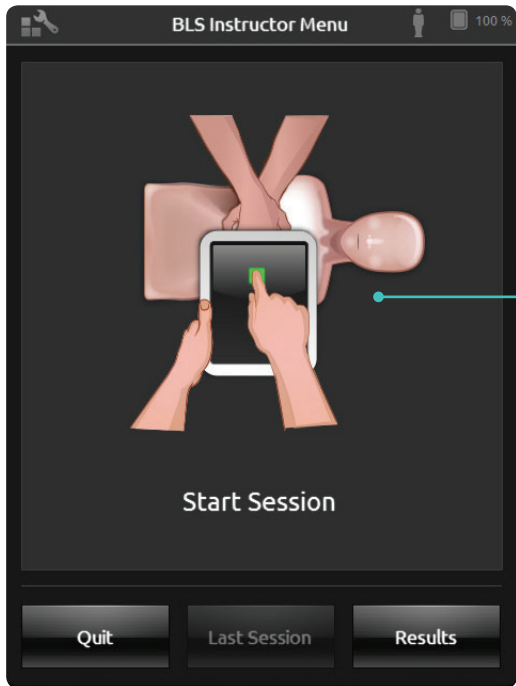


Monitor CPR classes with up to 6 manikins. You can have an overview of all the manikins or view details one by one.

* Depending on manikin model.

CHOOSE ALGORITHM


You can choose to practice **CPR** (both compressions and ventilations), **Compressions only**, or **Ventilations only**. For **CPR** sessions, select either **single rescuer** or **two-rescuer**. The difference between single rescuer and two-rescuer is in the scoring: participants in two-rescuer sessions are allowed less no-flow time than single rescuers are.

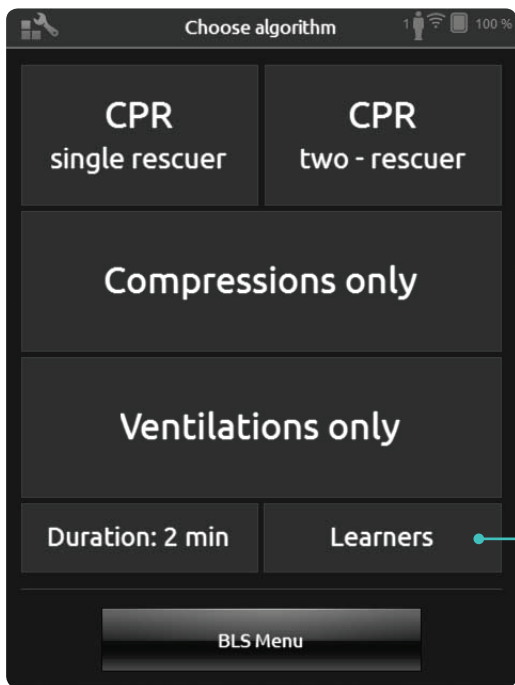

To assign learner names to connected manikins, tap **Learners**.


MANAGE LEARNERS

You can assign one or more learner names to each manikin. Tap **Learner** and assign the names either before, during, or after a session. The names are stored for later reference. View and edit the **Learners** list in **BLS Settings**.

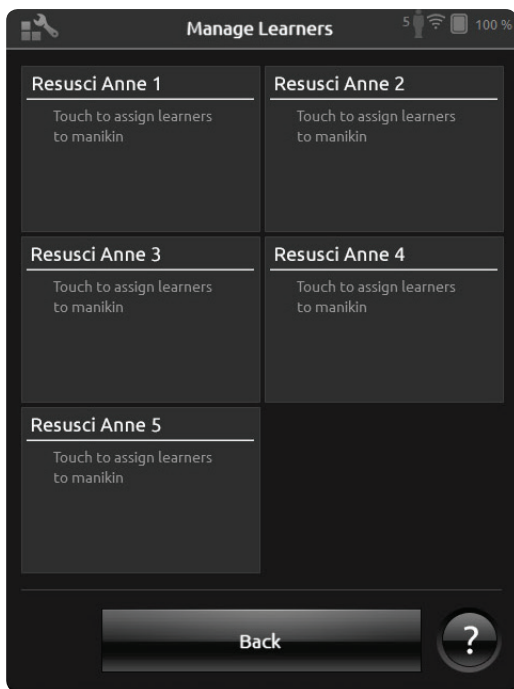


To manage a large group of learners, you can import a list of names to SimPad. Connect SimPad to a computer; using the SimPad-to-PC cable that comes with SimPad. SimPad appears as an external disk drive on your computer.

Create a document in “.txt” format named “class.txt” (using e.g. Notepad) that has one learner name per line. Transfer the file into the folder named “Learners” on the SimPad drive.

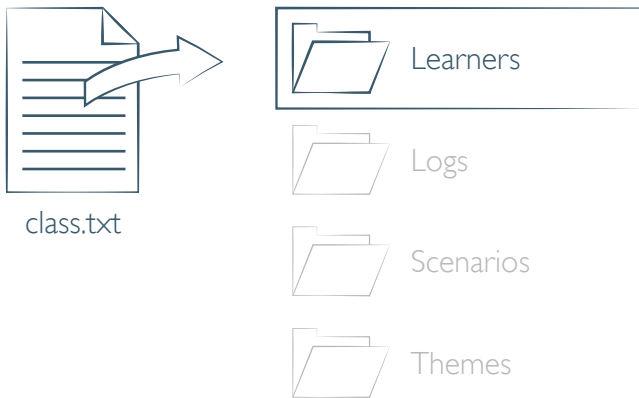


EVENT REGISTRATION

In **BLS Instructor**, you can register Events such as **Response Check** and **Call for help** during a session. These events are displayed in the timeline and stored in the session. You can create your own events from BLS Settings > View Event Registrations. The Event Registration feature can be disabled in BLS Settings.

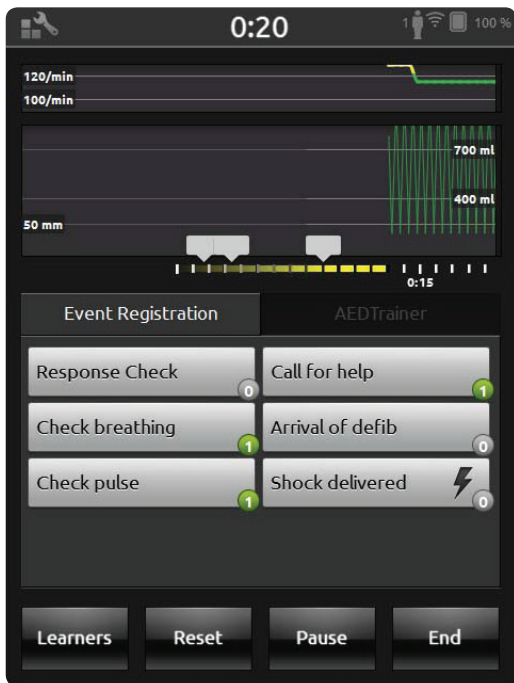

DEFIBRILLATOR FEATURES

When using Resusci Anne QCPR D you can give training using a live defibrillator.* SimPad recognizes this manikin automatically and allows you to select heart rhythms.

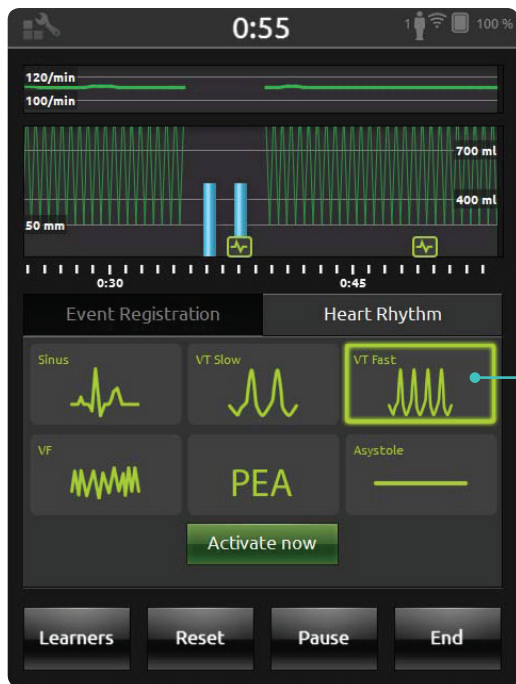


Active Rhythm

When a new rhythm is selected, it flashes to indicate that it's a pending rhythm. The rhythm is activated upon shock or when you press the **Activate now** button.

*You need a compatible adaptor cable to connect your defibrillator to the manikin.



SimPad SkillReporter

When using **Resusci Anne AED** you can give AED training with a Laerdal AED Trainer 2 or 3. Enable this feature in BLS settings by selecting your AED trainer type. When applicable training pads (Link Pads) are placed correctly on the manikin, SimPad can be used as remote control for the AED trainer:



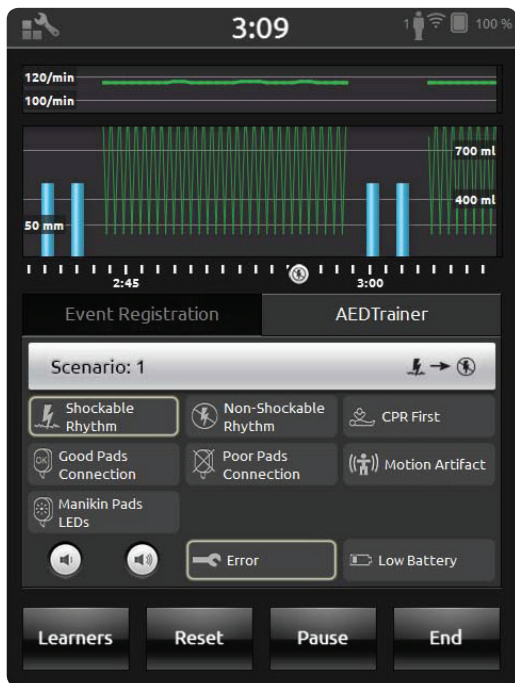


LIVE FEEDBACK

COMPRESSIONS

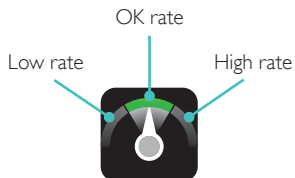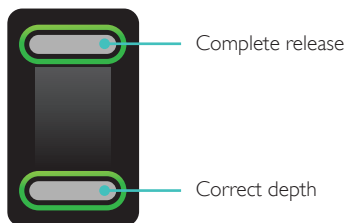


VENTILATION

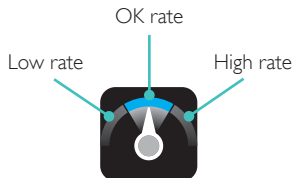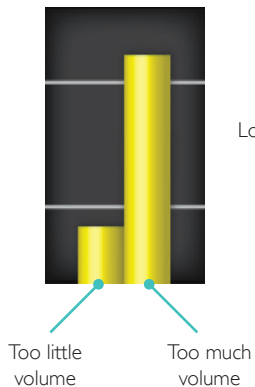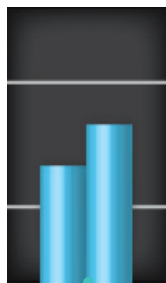


TIMELINE

Performance data for the previous 30 seconds is displayed in the timeline. The upper part of the timeline shows the compression rate curve. The lower part displays both the compression depth curve and the ventilation volume bars. When doing a **Ventilations Only** session, the ventilation rate curve is displayed instead of the compression rate curve. Events and AED activity is viewed in the timeline.

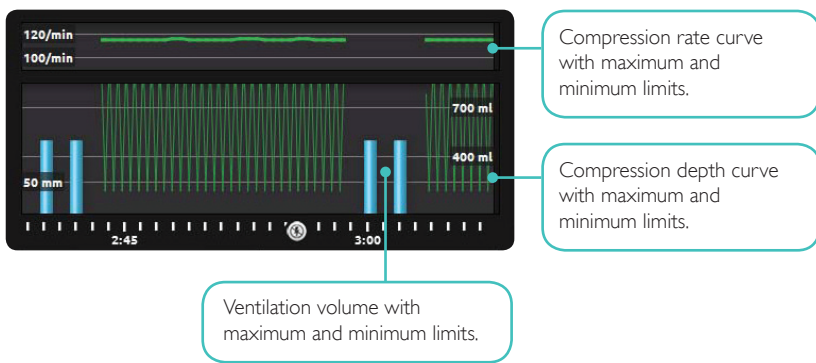


RESULTS

The first result screen gives you advice on how to improve your CPR performance. An overall score is displayed, if the overall score feature is enabled in BLS Settings. To get more details, tap or swipe the screen.

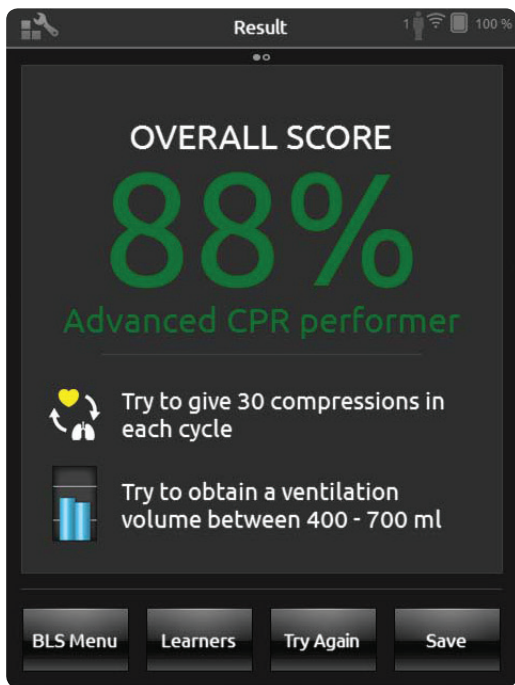


For more details on the scoring, please visit www.laerdal.com/qcpr.

The second result screen gives you more details. When you tap an area on the screen, an explanation of the data given there is displayed. To view compressions and ventilations, zoom in and scroll in the timeline.

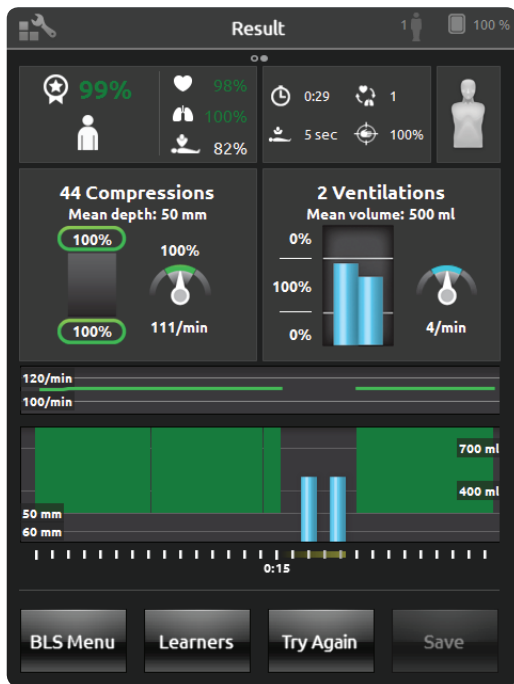


For scenario training with AED or when events are registered (Response Check, Check Breathing, etc.) the Flow Fraction is not included in the overall score. This is to allow for correct medical procedures, without having the overall score reduced. The Flow Fraction is then shown separately.

Events and AED actions are displayed in the timeline. Tap the events or heart rhythms for details.

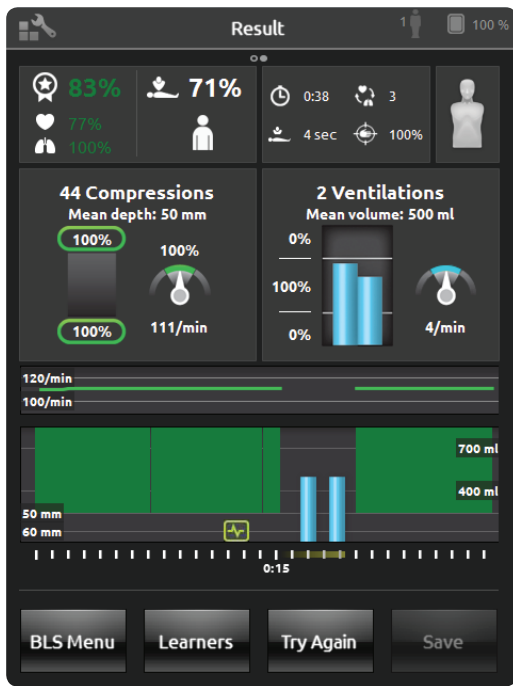


BLS SETTINGS

In the BLS Settings Menu, you can select predefined CPR Guidelines or specify your own custom thresholds, and more.

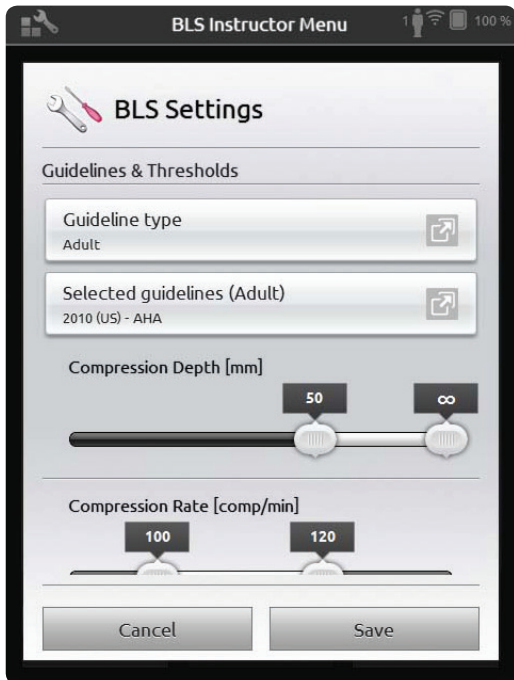

GET MANIKIN INFORMATION


To get detailed information about your manikin, tap and hold (2 seconds) on one of the connected manikins. This will toggle between manikin status (CONNECTED), manikin serial number (SNR), manikin software version (SW), and number of compressions performed on that manikin during its lifetime (COMP).

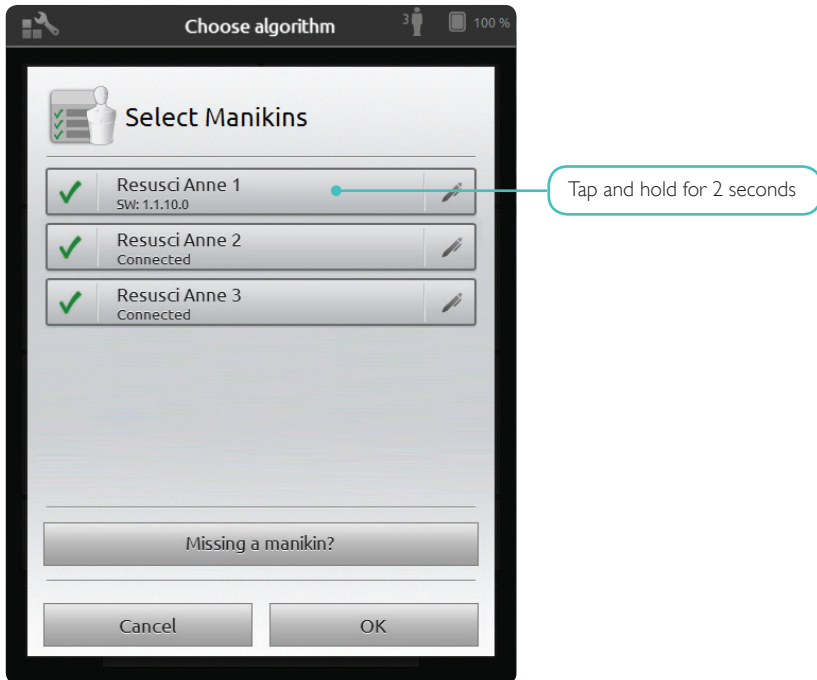

RED LIGHT ON RESUSCITATING MANIKIN

When also using SkillGuide, it's crucial that the cable is fully inserted. Otherwise the manikin can't communicate with SimPad SkillReporter. After inserting the cable, check the side panel. If the red light glows steadily, disconnect and reconnect the SkillGuide cable. Then switch the manikin off and on.

If the SkillGuide is disconnected and the red light still glows steadily after switching the manikin off and on, please refer to the manuals for the manikin or contact customer service for support.

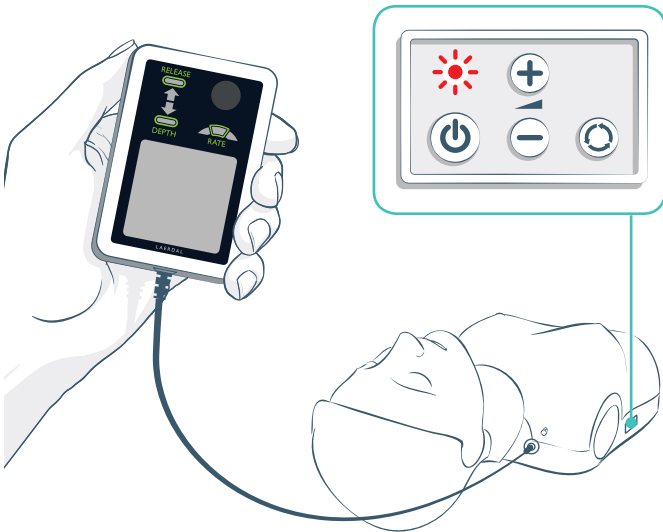

© 2013 Laerdal Medical AS. All rights reserved.
Device Manufacturer: Laerdal Medical AS, P.O. Box 377,
Tanke Svilandsgate 30, 4002 Stavanger, Norway
T: (+47) 5151 1700

20-06569 Rev D

