

ASSEMBLY INSTRUCTIONS (USE AND CARE SHEET)



TRAINING MANIKIN

THANK YOU FOR INVESTING IN CPR PROMPT® PROFESSIONAL TRAINING PRODUCTS

If you have any questions about your manikins,
or need to order extra lung bags:

Please call us **TOLL FREE** at 1-800-558-9595

Write to:

Nasco

901 Janesville Avenue

Fort Atkinson, WI 53538-0901

E-mail: lifeform@eNasco.com

Website: www.cprprompt.com


TO START...

- Use this sheet for instructions on how to assemble your manikins.
- Clean the manikin heads before you start (refer to cleaning instructions at the end of this sheet).
- Use only “foam manikin” AED training pads (refer to part numbers at the end of this sheet).
- If you have any problems with assembling these manikins, refer to the troubleshooting guide located at the end of this sheet (or call us toll free at 1-800-558-9595).


CONTENTS & PARTS

Each manikin comes with a supply of face shield lung bags and an insertion tool.

ADULT/CHILD MANIKIN

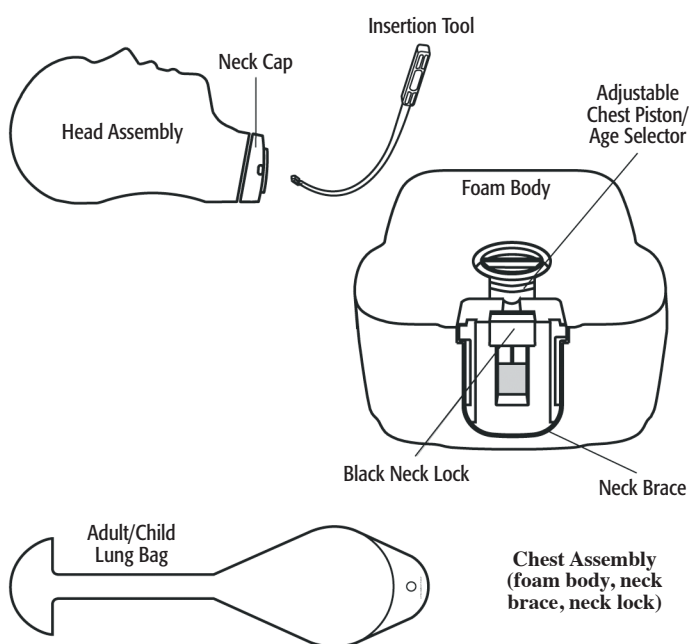

INFANT MANIKIN

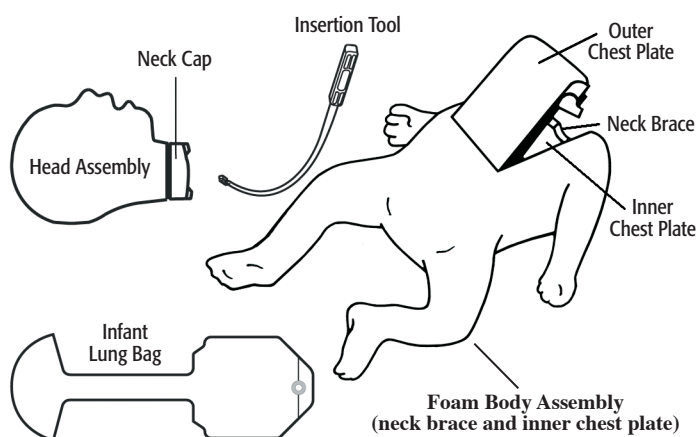


TIP:

Before installing lung bag — Wrinkle the neck and pouch of the bag by rolling it between your thumb and fingers. This will help ensure that the air flows freely into the lung bag during use.

ADULT/CHILD ASSEMBLY


1

- Feed the insertion tool through the neck and up so hook extends out of the mouth.

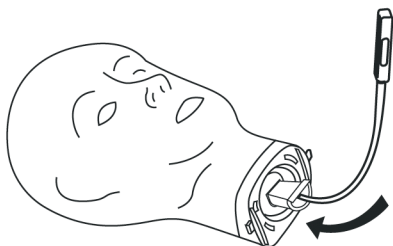


2

- Place bag on face of manikin.
- Attach hook of tool to bottom hole on bag.

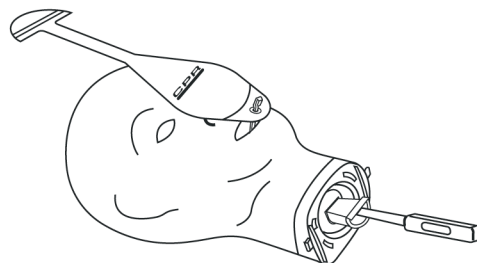


3

- Gently pull insertion tool until the bag extends out of the neck opening. Face shield portion should cover mouth.

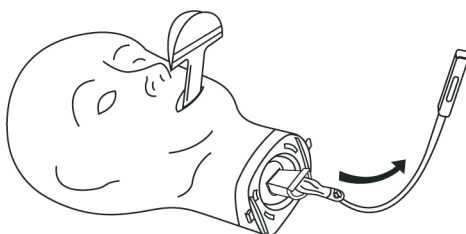


4

- Remove adhesive strip and fold tabs over mouth.
- Attach head to torso.

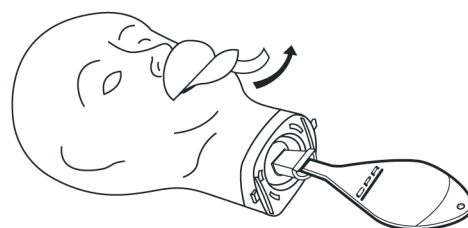


5

- Pull the piston out of the bottom of the chest cavity.

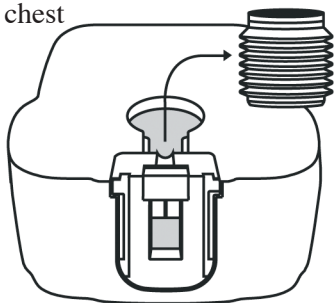


6

- Remove the black neck lock from the neck brace.
- Squeeze the top of the black neck lock and pull up and out of the neck brace.

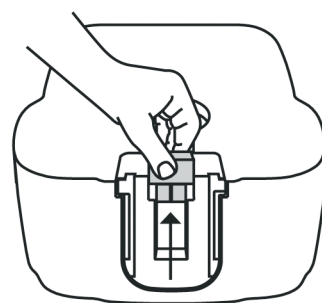


7

- While holding the head, pull the lung bag through the opening in the neck brace and lay the bag flat, centered in the chest cavity.

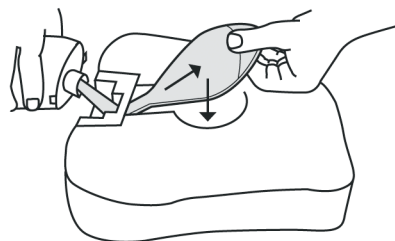


8

- With the head facing down, line up the two pins on the head with the two slots in the neck brace. Push the pins until they slide into the slots and the head drops down into place.

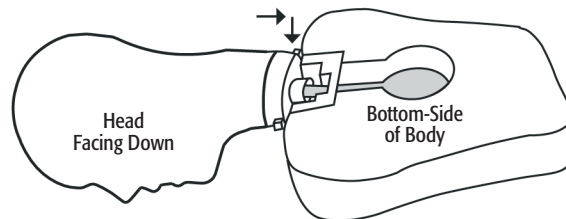

9



- Replace the black neck lock in the neck brace to hold the head in place.

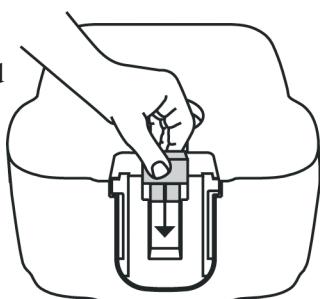

Make sure that there are no kinks in the lung bag.

10

- a) Turn the age selector dial on the chest piston so the pin is next to the ADULT or CHILD arrow (whichever age you are learning).
- b) Replace the chest piston in the chest cavity so that the age selector dial is visible in the body.

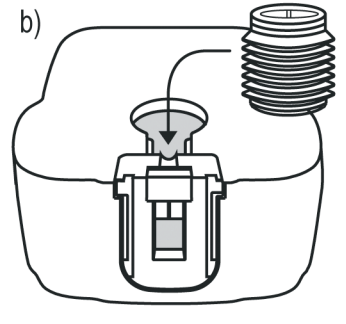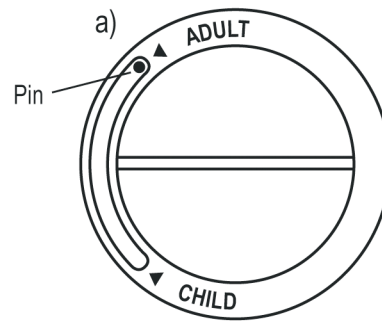

INFANT ASSEMBLY


1

- Feed the insertion tool through the neck and up so hook extends out of the mouth.




2

- Place bag on face of manikin.
- Attach hook of tool to bottom hole on bag.

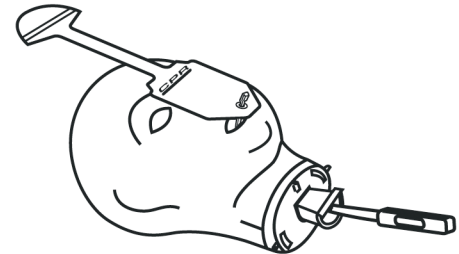

3

- Gently pull insertion tool until the bag extends out of the neck opening. Face shield portion should cover mouth.

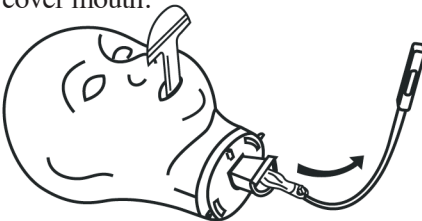


4

- Remove adhesive strip and fold tabs over mouth.
- Attach head to torso.




5

- Open the infant chest.
- Grip the collar of the outer chest plate and lift up.

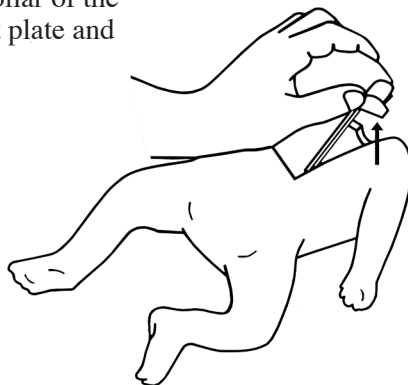


6

- Slide the head and lung bag into the neck brace.
- With the head facing up, line up the two pins on the head with the slots in the neck brace.
- Guide the pins into the slots and let the head drop down.

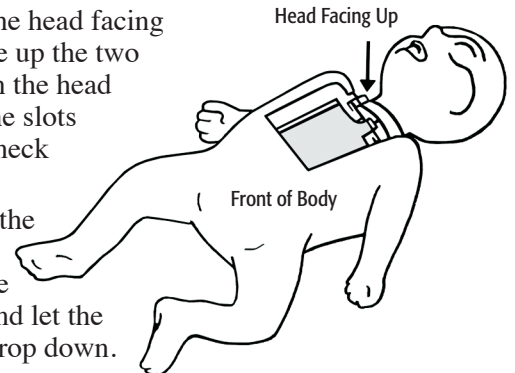


7

- Center the lung bag on top of the inner chest plate.
- Make sure that the bag is laying flat, and there are no "kinks."

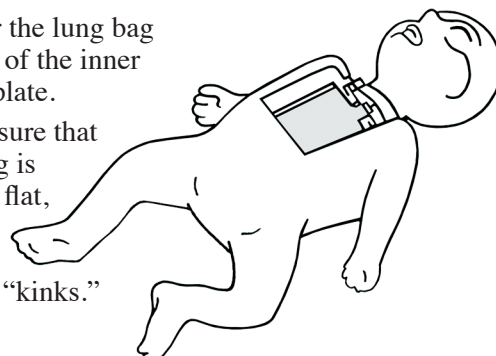

8

- Replace the outer chest plate.

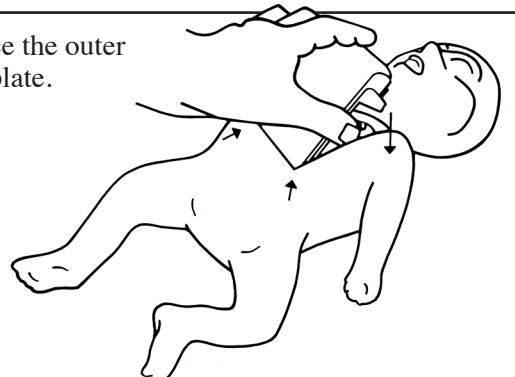

TROUBLESHOOTING GUIDE

Some suggestions if you encounter any of these problems...

If you cannot get the chest to rise when you blow, make sure that:

- The head is tilted back.
- The nostrils are covered.
- The lung bag is properly positioned: it is not twisted and is laying flat inside the chest cavity.
- The lung bag is properly attached to the head.
- The bag has been fed through the neck cap.
- The bag has not been torn during assembly.
- The neck cap is in the correct position (“FRONT” of neck cap should be lined up with the front of the head.)

If you do not hear a click when you compress the chest of the Adult/Child Manikin:

- Make sure that you are pressing deep enough (you should hear the click when you compress the foam about one third of its depth).
- Check the Adjustable Chest Compression Piston and make sure that the black pin in the Age Selector Dial is pushed all the way over to the Adult or the Child — depending on the age you are learning.

CLEANING INSTRUCTIONS

NECK CAP

Before cleaning heads, remove neck cap:

- To remove the neck cap, twist the neck cap counter-clockwise like the lid on a jar (about a quarter of a turn).



HEADS

Heads can be cleaned by the following methods:

- Run them through a dishwasher, OR...
- Wash heads with soap and water. Then soak 10 minutes in a solution of ¼ cup of bleach mixed with one gallon of water, OR...
- Sanitize with alcohol wipes.
- After cleaning head, replace the neck cap by twisting the neck cap back onto the head. Make sure the “FRONT” of the neck cap is lined up with the front of the head.

BODIES

- The bodies can be safely cleaned with a bleach/water solution (see details under “HEADS” section), alcohol, or any other sanitizing agents.*
- * Manikin bodies are foam based and are not waterproof. They should not be submerged in water.

AED TRAINING

- Use only “foam manikin” AED training pads:
 - **LF06501U** Medtronic Physio-Control Style
 - **LF06502U** Heartstream Style
 - **LF06503U** Survivalink Style
- Peel pads off gently after each training session, being careful not to rip the skin coating.

OTHER AVAILABLE CPR PROMPT® PRODUCTS

LF06928U CPR Prompt® Small Blue Bag (17" x 8½" x 11¼")
LF06929U CPR Prompt® Large Blue Bag (30" x 13" x 19")
LF06940U CPR Prompt® Small Gray Bag (17" x 8½" x 11¼")
LF06941U CPR Prompt® Large Gray Bag (30" x 13" x 19")
LF06943U Insertion Tool — 5-Pack, Adult/Child/Infant

LF06923U Black Neck Lock
LF06206U Adult/Child Face Shield/Lung Bags – 10-Pack
LF06208U Adult/Child Face Shield/Lung Bags – 100-Pack
LF06207U Infant Face Shield/Lung Bags – 10-Pack
LF06209U Infant Face Shield/Lung Bags – 100-Pack

CPR BLUE AND TAN ADULT/CHILD MANIKIN PART NUMBERS

LF06911U Blue Head Assembly
LF06918U Blue Neck Cap
LF06932U Blue Chest Assembly (foam body, neck brace, neck lock)
LF06913U Adjustable Chest Piston/Age Selector

LF06910U Tan Head Assembly
LF06919U Tan Neck Cap
LF06931U Tan Chest Assembly (foam body, neck brace, neck lock)
LF06909U Replacement Clicker Blades (Pkg. of 5)

CPR BLUE AND TAN INFANT MANIKIN PART NUMBERS

LF06915U Blue Head Assembly
LF06926U Blue Neck Cap
LF06922U Blue Outer Chest Plate
LF06925U Blue Foam Body Assembly (neck brace & inner chest plate)
LF06935U Blue Body and Chest Assembly (foam body, chest, inner & outer chest plates, neck brace)

LF06914U Tan Head Assembly
LF06927U Tan Neck Cap
LF06920U Tan Outer Chest Plate
LF06924U Tan Foam Body Assembly (neck brace & inner chest plate)
LF06936U Tan Body and Chest Assembly (foam body, chest, inner & outer chest plates, neck brace)