



Basic Child **CRiSis**[™] Manikin LF03765U Instruction Manual



WARNING: Products may contain dry natural rubber.

Life/form[®] Products by Nasco



Table of Contents	Page
Introduction	2
Assembly of Your Basic Child CRiSis ™ Manikin	2
About the Basic Life/form ® Child Airway Management Trainer Head	3
Additional Supplies to Upgrade the Manikin	4

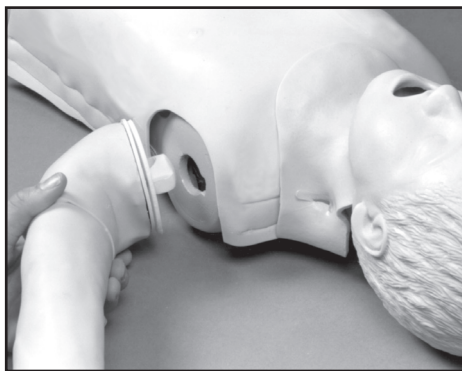


Figure 1

Basic Life/form® Child CRiSis™ Manikin

This full-body child manikin has been created to offer realistic training and practice of child airway management and CPR education. The Basic Life/form® Child CRiSis™ Manikin is anatomically correct in detail and represents the size of an eight-year-old child. Realistic anatomy and landmarks include a mouth, tongue, oral pharynx, larynx, epiglottis, arytenoids, vocal cords, trachea, and esophagus. In addition to the palpable and visual landmarks, the head, neck, and jaw are fully articulated, allowing a realistic jaw thrust feature. This manikin has separate left and right lungs for auscultation; has oral, nasal, and digital intubation capabilities; and includes an inflatable stomach bladder to indicate esophageal intubation. Also features E.T., E.O.A., P.T.L., L.M.A., E.G.T.A., and Combitube® insertion. This basic airway management trainer may be gradually updated to the Life/form® Child CRiSis™ or Deluxe Child CRiSis™ manikins by adding the available upgrades and the Interactive ECG Simulator (LF03670U). Includes pump spray lubricant. See page 4 for complete list of modular update components. Three-year warranty.

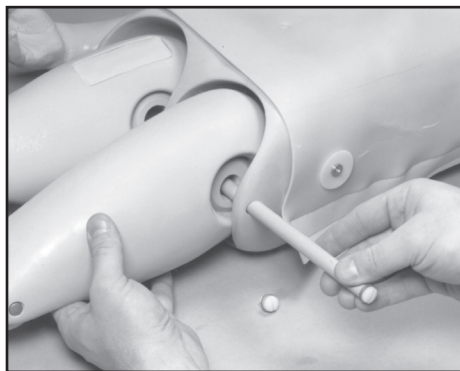


Figure 2

Assembly of Your Basic Child CRiSis™ Manikin


To facilitate packing, your manikin comes with the arms and legs unattached.

To connect the arms to the torso, place each in a position such that the hand is above the head. **(See figure 1.)**

Align the arm fitting with the torso socket and plug the arm into place. Finish by rotating the arm forward so it lies along the side of the body.

To connect the legs to the torso, simply slide the waist pin (supplied) through the torso and legs, pinning the legs in their correct position. **(See figure 2.)**

The Airway Management Trainer

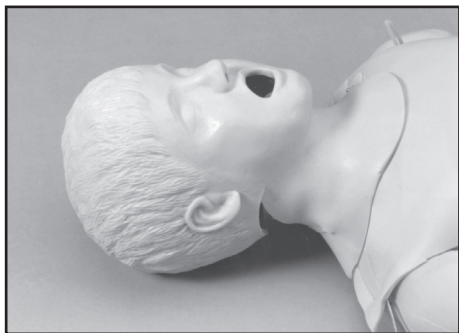

About the Simulator

The Basic **Life/form**® Child Airway Management Trainer Head is the most realistic simulator available for the training of intubation and resuscitation skills. Nasco has taken great care to create a trainer that is anatomically correct in respect to both size and detail.

Landmarks include nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true vocal cords, cricoid ring, tracheal rings, trachea, esophagus, lungs, and stomach.

The Basic **Life/form**® Child Airway Management Trainer Head allows you to practice oral, digital, and nasal intubation, as well as CPR. Suction techniques can also be performed and evaluated. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D.

List of Components

- Child Airway Management Trainer Head
- Right and Left Lung Simulators
- Stomach Simulator
- Pump Spray Lubricant
- 3 Nylon Hose Clamps



Figure 3

Lubrication

Before using the Basic **Life/form**® Child Airway Management Trainer, lubricate both the oral cavity and the endotracheal tube with the pump spray lubricant provided. (**See figure 3.**) **Note:** Nasco recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Child Trainer. The use of a silicone-based or similar lubricant may seriously damage the simulator, thus voiding Nasco's warranty on the trainer.

Cleaning the Face

Normal soil can be removed from the face with mild soapy water. Nasco Cleaner (LF09919U) will remove stubborn stains. Simply apply Nasco Cleaner to soiled area and wipe clean with a soft cloth or paper towel.

Note: Avoid using cleaner around mouth area if students will be applying direct mouth-to-mouth resuscitation techniques, as the cleaner may be toxic if ingested. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ball-point pens, will transfer indelible stains. Do not use any cosmetics.

Available Supplies

- LF03631U** Replacement Headskin with Airway
- LF03627U** Replacement Lungs
- LF03628U** Replacement Stomach
- LF03644U** Pump Spray Lubricant
- LF09919U** Nasco Cleaner

Update Your Basic *Life/form*® Child *CRiSis*™ Manikin with *Life/form*® Modular Components

LF03611U Defibrillation Chest Skin

LF03612U Injectable Training Arm

LF03613U Blood Pressure Arm

LF03614U Intraosseous Infusion/Femoral Access Leg

LF03618U Intraosseous Replacement Bones

LF03626U Femoral Replacement Injection Pad

LF03619U Intraosseous Replacement Skin

LF03683U Intraosseous Infusion Needle

LF03628U Replacement Stomach

LF03627U Replacement Lung Set Only

LF03670U Interactive ECG Simulator

Other Available *Life/form*® Simulators

LF00698U	Adult Injectable Arm (Light)	LF01174U	NG Tube & Trach Skills
LF00855U	Male Catheterization	LF01184U	Venatech IM & Sub Q
LF00856U	Female Catheterization	LF01193U	Special Needs Baby
LF00901U	Prostate Examination	LF03000U	CPARLENE ® Series
LF00906U	Ostomy Care	LF03601U	Adult Airway Management Trainer with Stand
LF00929U	Surgical Bandaging	LF03602U	Adult Airway Management Manikin
LF00957U	Enema Administration	LF03609U	Child Airway Management Trainer with Stand
LF00958U	Pediatric Injectable Arm	LF03616U	Child CRiSis ™ Manikin
LF00961U	Intramuscular Injection	LF03617U	Deluxe Child CRiSis ™ Manikin with Arrhythmia Tutor
LF00984U	Breast Examination	LF03620U	PALS Update Kit
LF00995U	Arterial Puncture Arm	LF03623U	Infant Airway Management Trainer with Stand
LF00999U	Pediatric Injectable Head	LF03632U	Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF01005U	First Aid Arm	LF03633U	Child Airway Management Trainer Torso
LF01008U	Intradermal Injection Arm	LF03693U	Basic Buddy ® CPR Manikin
LF01012U	Heart Catheterization (TPN)	LF03699U	"Airway Larry" Airway Management Trainer
LF01019U	Ear Examination	LF03709U	Infant CRiSis ™ Manikin
LF01027U	Peritoneal Dialysis	LF03720U	Baby Buddy ™ Infant CPR Manikin
LF01028U	Suture Practice Arm	LF03750U	Bariatric CPR Manikin
LF01034U	Suture Practice Leg	LF03770U	Chest Tube
LF01036U	Spinal Injection	LF03953U	CRiSis ™ Manikin, Complete
LF01037U	Hemodialysis Practice Arm	LF03955U	Deluxe CRiSis ™ Manikin
LF01038U	Episiotomy Suturing Set	LF03956U	Deluxe "Plus" CRiSis ™ Manikin
LF01042U	Suture Kit	LF03965U	Adult CRiSis ™ Auscultation Manikin
LF01062U	Pelvic, Normal & Abnormal	LF03966U	Adult CRiSis ™ Auscultation Manikin with ECG Simulator
LF01063U	Stump Bandaging, Upper	LF04000U	GERI ™/ KERI ™ Manikin Series
LF01064U	Stump Bandaging, Lower	LF04200U	Adult Sternal Intraosseous Infusion
LF01069U	Cervical Effacement	LF06001U	CPR Prompt® Adult/Child Manikin
LF01070U	Birthing Station	LF06012U	CPR Prompt® Infant Manikin
LF01082U	Cricothyrotomy	LF06200U	CPR Prompt® Keychain Rescue Aid
LF01083U	Tracheostomy Care	LF06204U	CPR Prompt® Rescue and Practice Aid
LF01084U	Sigmoidoscopic Examination		
LF01087U	Central Venous Cannulation		
LF01095U	Blood Pressure Arm		
LF01108U	Infant Intraosseous Infusion		
LF01121U	Advanced IV Arm		
LF01131U	Venipuncture and Injection Arm		
LF01139U	Advanced IV Hand		
LF01142U	Auscultation Trainer		
LF01143U	Testicular Exam		
LF01152U	Male & Female Catheter		
LF01155U	Advanced CPR Dog		
LF01162U	Venatech IV Trainer		

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1.800.558.9595

eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 2008 PRINTED IN U.S.A. NP 117-08/RV 9-15