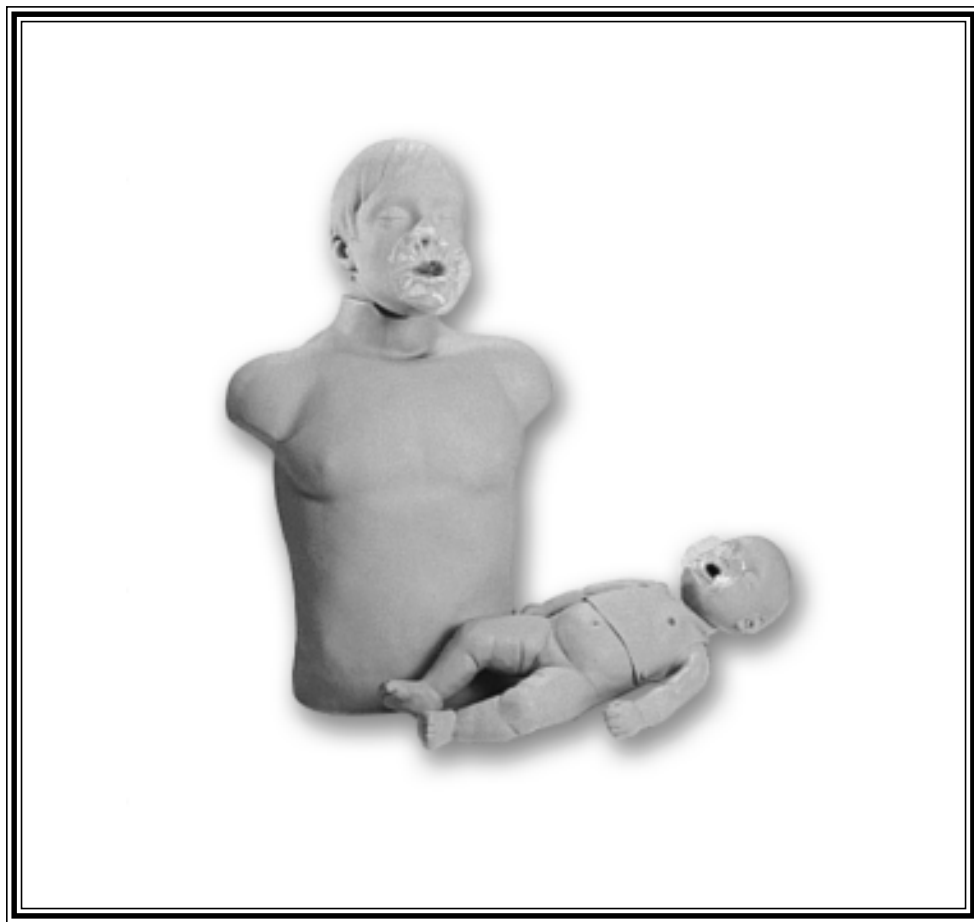

Sani-Baby, Sani-Man & Sani-Child Instruction for Use



 **Simulaids**

Thank you for buying an American made product.

The purpose of this instructional is to familiarize the user with the function, care and maintenance of the Simulaids' Sani-Baby training manikin. It is not meant to provide instructions in CPR or other emergency life procedures. Please read it thoroughly. All Simulaids manikins are designed and manufactured to exacting standards and will provide years of effective and trouble-free use when the simple steps of care and maintenance are regularly performed.

Note: The manikin you have just purchased is a member of Simulaids' Sani-Man line. These manikins serve many individuals during a training session. In order to eliminate cross-contamination, or the possibility of cross-contamination, these manikins are designed with a "single use airway." That means that only one person uses the plastic bag airway system.

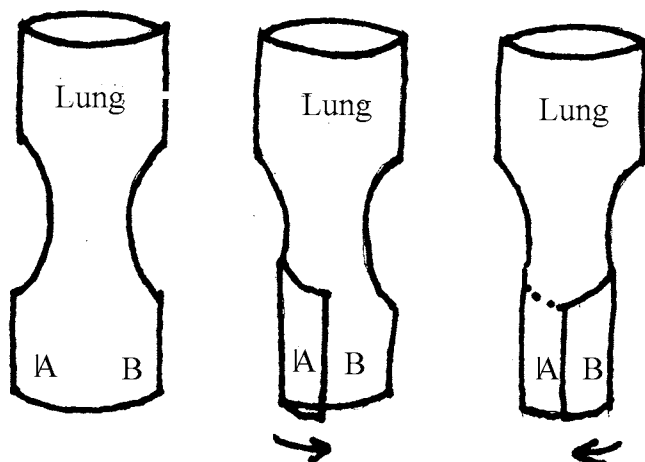
When the individual completes his/her airway inflations, the bag is to be removed and the next student will place his/her airway in the manikin. The airway can be retained by an individual for further use during that training session, but the airway should be discarded at the end of that session. This is the only recommended procedure for eliminating cross-contamination.

Airway Removal

1. Remove chest overlay from shoulders and peel down.
2. Remove the face shield bag by pulling it out of the mouth.

Airway Installation

1. Fold opened end of face shield bag as shown below. Tilt head back and place folded end up through the mouth.
2. With head tilted back lay the lung flat against the chest and pull the overlay back up into place and snap.
3. Form the face shield to cover face from nose to chin.



WARRANTY: Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required.

RETURN POLICY: Should it be necessary to return an item for credit for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.

<u>Part #</u>	<u>Description</u>
2121	Sani-Baby
2124	Sani-Baby 4-Pack
2125	Sani-Baby Face Shield Lung System (100)
2252	Baby Carry Bag
2260	Baby Carry Bag 4-pack
2140	Sani-Child
2131	Sani-Man
2141	Sani-Child Face Shield Lung System (100)
2135	Sani-Man Face Shield Lung System (100)
2252	Carry Bag Torso Manikin (Sani-Child)
2526	Carry Bag Torso Manikin (Sani-Man)



 **Simulaids**



PO Box 1289 - 16 Simulaids Dr - Saugerties NY 12477
(845)-679-2475

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com

4/2006